

use BOViTAG™ to identify:

- dry / lame cows
- breeding / reproduction
- sorting / culling
- medical treatment / risk management

By writing with a permanent marker on the tag, you can quickly identify the reason why the cow was tagged.


BOViTAG™ is self-adhesive and is available in various colours.

APPLYING BOViTAG™

WRAP THE TAG AROUND THE TAIL

Choose application area towards the lower end of the tail. Fold the tag around the tail, so that the two edges of the tag joins again. Alternatively you can wrap it around the tail. Trim the tag if desired.

Do not fit it too tightly around the tail, since it might affect circulation.

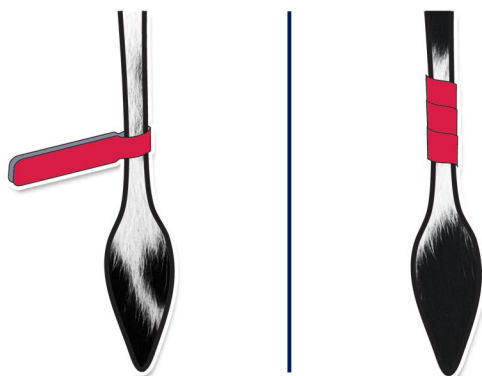

WRAP THE TAG AROUND THE LEG



Wrap the BOViTAG™ around the lower end of the leg.

Do not fit it too tightly, since it might affect circulation.

REMOVING THE TAG

Cut the tag with a pair of scissors or knife.

Let us know what you think!

We are always striving to improve our product. We see you as an essential part of this process and value your feedback.
feedback@bovertime.com

Warranty Statement

This product is guaranteed free of defects in material and workmanship for two years from date of manufacture. Suitability of the product for the Buyer's intended purpose is the responsibility of the Buyer. Liability is limited to the replacement value of the product.



Visit www.bovertime.com for more information or contact us at info@bovertime.com



Made in South Africa